

## Applying the 3pp Prima Thumb Brace



Slide thumb into the brace



Close the thumb strap with light tension



Hold the flap in place as you grasp and attach the Easy-On Tab



Grasp the strap or the edge of the flap and pull the brace closed



Stretch the strap lightly and bring the strap around the base of your thumb



Continue lightly stretching the strap around your thumb and over your web space



Bring the strap around the base of your thumb and attach the tab



The bendable plastic stay can be removed from the interior pocket as needed

### *What makes this orthosis a better option for treating my patient's thumb pain?*

**#1 Comfort** - Non-bulky, foam-lined material has natural memory so straps do not need to be pulled tight for a comfortable, customizable support. Binding eliminates fraying and breathable material helps dissipate heat.

**#2 Fit** - Straps attach anywhere on the loop material for a fully adjustable fit. Easy-On Tab makes applying the splint easy, and the wrap around thumb strap conforms around the CMC and MCP joints without restricting motion.

**#3 Function** - Carefully designed to conform in the palm so fingers are free for unimpeded function. The contoured thumb strap supports the CMC and MCP joints when pinching or gripping.

### To Order from 3-Point Products

Health Care Providers

888-378-7763

Mon-Fri 8:30am - 5:00pm EST

service@3pointproducts.com

### Online Ordering

Patients & Consumers

www.ohmyarthritis.com



Scan Here