

Application Instructions



Slide splint onto thumb



Close strap to comfort



Contour to fit thumb

FAQs

What makes this orthosis a better option for treating CMC Thumb Arthritis or Gamekeepers and Skier's Thumb?

1. **Comfort** - Offers minimal wrist restriction and is extremely lightweight. Made with 3 layers of soft cushioning and smooth velour straps that won't catch on clothing.
2. **Fit** – Fully customizable orthosis extends just below the wrist for firm support of the CMC and MP joints. The thin, lightweight interior frame can be contoured around the hand and end joint of thumb for an optimal fit. Can be worn on right or left hand, helping with economic stocking in the clinic.
3. **Function** - Easy to apply and wear for extended periods of time. Open palm design allows fingers to move.

Cleaning Instructions

Clean regularly using a damp sponge or cloth and a small amount of gentle detergent. Rinse with cold water, use a towel to absorb water and lay flat or hang to air dry. Do not place in dryer. 100% Latex free.

Suggested Billing Code

HCPCS code L3807 Use of this suggested code does not guarantee insurance reimbursement for this orthosis. Please consult with your medical billing specialist for assistance with billing questions. Code is current as of January 2020.

flip the page 