

3-Point Products,® Inc.



Instructions For Use - English (EN)



www.3pp-eifu.com

Description: Oval-8 Finger Splints are a molded plastic device designed to fit on a digit for support, protection and alignment. They are available in 14 sizes to accommodate a range of finger sizes. Oval-8 Finger Splints are waterproof and can be worn in wet conditions.


Intended Use: Oval-8 Finger Splints are designed to apply three points of control around a digit to hold a knuckle straight, prevent or limit movement for protection and to allow healing. The device can be used to manage multiple finger conditions depending on where and how it is worn on the digit.

Precautions: Not for use on open wounds. Not for use if swelling is present. Not for use on toes. The device may cause irritation or pressure on the digit. Discontinue use if irritation or pressure occurs. Contact your health care provider if irritation persists. Follow cleaning guidelines on a regular schedule. **Caution:** Not to be worn by children under 5 years old. May cause choking if swallowed.




Direction: How to wear Oval-8 Finger Splints to manage different finger conditions.

1. Boutonniere Deformity

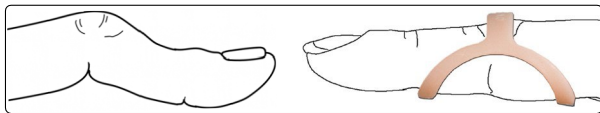


4. Mallet Finger

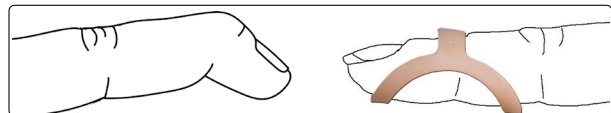


2. Crooked Finger




5. Swan Neck Deformity

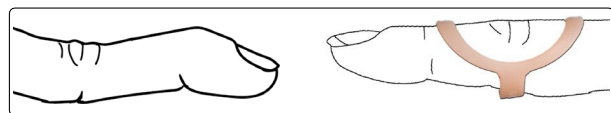



3. Trigger Thumb

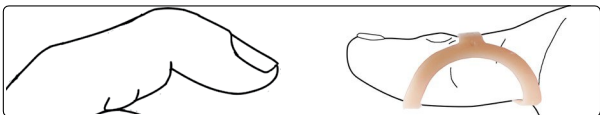

6. Trigger Finger

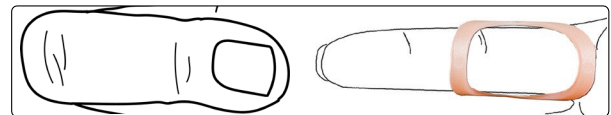

Note: Oval-8 splints fit tighter or looser depending on which end is put on the finger first. The plus + sign indicates the looser end.

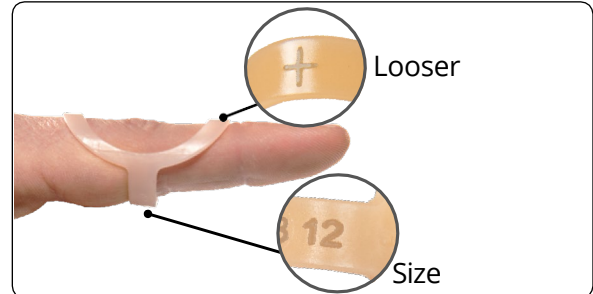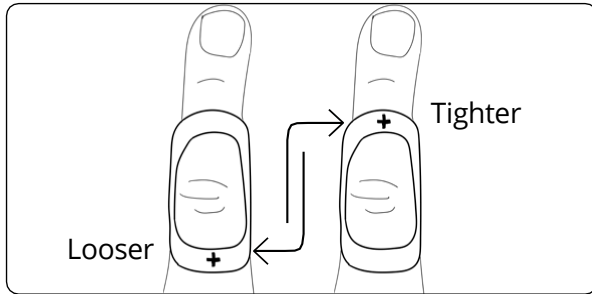

Measuring for an Oval-8 Finger Splint: Instructions for measuring digits for the correct size of product can be found on pages 3 & 4.

How Supplied: Product is manufactured with pure polypropylene. The product is latex-free. Do not use if the device is damaged. Store in a cool, dry place. It is available in 14 sizes, packaged in various quantities. Packaging options are on page 4.

Cleaning: Hand wash with soap and water, and air dry the product regularly. Disinfect the product with 70% isopropyl alcohol or a 10% bleach solution.

Disposal: The product is intended to be reusable on the same person multiple times. Discard the device if it shows signs of excessive wear and replace with a new product as necessary.


When to Consult a Health care Professional: Consult a health care professional if experiencing any adverse reaction to the device including skin reaction or changes in condition.

Notice to User and/or Patient: In the event of an adverse incident related to the device, remove the device and report the incident to the distributor and the Authorized Representative in the Member State in which the user resides.

Distributors: www.3pp-eifu.com/#distributors

Label Symbols: All of the symbols used on the product label or product literature are defined in the 3-Point Products Symbols Legend here: www.3pp-eifu.com/#symbols



3-Point Products, Inc.
118 Log Canoe Circle
Stevensville, MD 21666 USA
www.3pointproducts.com
+001.410.604.6393 



Sallmann Medical GmbH
Alois-Stockinger-Str.16
5020 Salzburg Austria
service@sallmann-medical.eu



Swiss AR Services GmbH
Industriestrasse 47
CH-6300 Zug

The Original






Finger Splint

Measuring your Finger(s) for an Oval-8 Finger Splint

Oval-8 Finger Splints are a medical device and a snug fit generally provides the most effective results. The correct size depends on what the finger problem is and where the band will be worn. Please follow instructions

You will need:

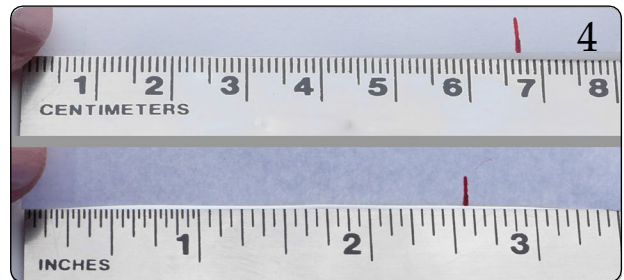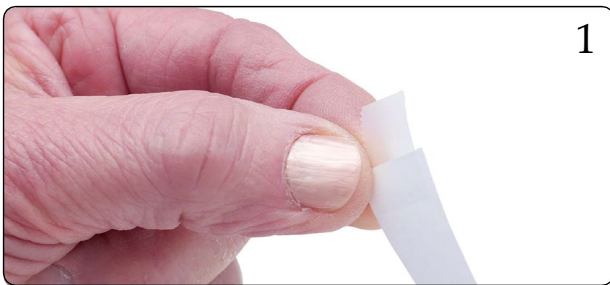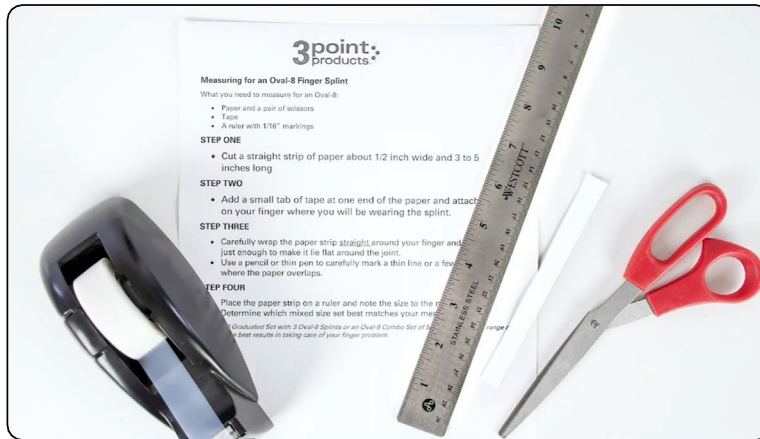

The Original



Finger Splint

Oval-8 Sizing & Packaging Charts

Oval-8 Size Chart (English)		
Oval-8 Size	Measurement Inches	Measurement Centimeters
2	1-3/4"	4.3
3	1-13/16"	4.5
4	1-15/16"	4.95
5	2-1/6"	5.27
6	2-3/16"	5.7
7	2-5/16"	5.86
8	2-7/16"	6.14
9	2-1/2"	6.35
10	2-11/16"	6.82
11	2-13/16"	7.09
12	2-15/16"	7.4
13	3"	7.65
14	3-3/16"	8.09
15	3-1/4"	8.2

Oval-8 Packaging Chart							
Oval-8 Size	One (1) per package	Three (3) per package	Five (5) per package	Graduated Set three (3) sizes	Combo Set five (5) sizes	Oval-8 Kit Clinic box with 44 splints Quantity per size	
2	✓	✓	✓	2-3-4		Size 2 & 15	1
3	✓	✓	✓	4-5-6		Size 3-4 & 13-14	2
4	✓	✓	✓	6-7-8		Size 11 & 12	3
5	✓	✓	✓	8-9-10		Size 5 & 10	4
6	✓	✓	✓	10-11-12		Size 6-7-8-9	5
7	✓	✓	✓	13-14-15			
8	✓	✓	✓		2-3-4-5-6		
9	✓	✓	✓		5-6-7-8-9		
10	✓	✓	✓		8-9-10-11-12		
11	✓	✓	✓		11-12-13-14-15		
12	✓	✓	✓				
13	✓	✓	✓				
14	✓	✓	✓				
15	✓	✓	✓				
Sizing Set one (1) each size - 14 total 2-3-4-5-6-7-8-9-10-11-12-13-14-15							
All packages may not be available from all sellers							