

3-Point Products,[®] Inc.



CE

Instructions For Use - English (EN)



www.3pp-eifu.com



Description: A 3pp[®] Toe Loop[®] supports and protects injured digits.

Intended Use: A flexible device designed to secure an injured digit to an adjacent digit for protection, to hold digits in proper alignment, facilitate motion and aid in healing.



Precautions: Not for use on open wounds. The device may cause skin irritation. Discontinue use if irritation occurs. Contact your health care provider if irritation persists. Follow cleaning guidelines and apply a clean wrap on a regular schedule. Discuss with healthcare provider prior to use if you have diabetes or reduced circulation.

Directions: How to apply

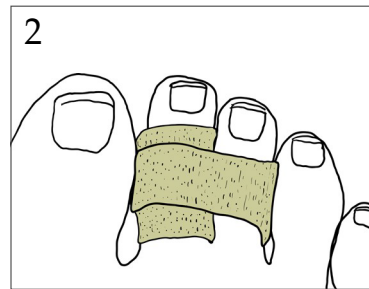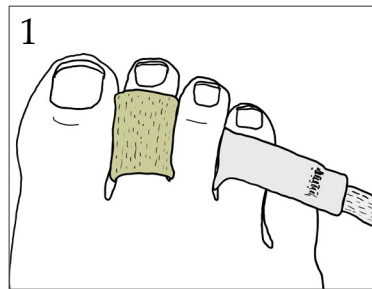

Narrow 7/8" x 2 3/8" (2.2cm x 6.0cm)



Wide 1 1/4" x 4 1/4" (3.1cm x 10.8cm)



How Supplied: The product is manufactured out of nylon, Lycra[®] and foam. Materials are latex-free. It is available in three sizes, packaged in various quantities. Do not use if the device is damaged. Store in a cool, dry place.

Cleaning: Hand wash and air dry the product regularly. Apply a clean product as needed according to conditions.

Disposal: The product is intended to be reusable on the same person multiple times. Discard the device if it shows signs of excessive wear and replace with a new product as necessary.


When to Consult a Healthcare Professional: Consult a healthcare professional if experiencing any adverse reaction to the device including skin reaction or changes in condition.

Notice to User and/or Patient: In the event of an adverse incident related to the device, remove the device and report the incident to the distributor and the Authorized Representative in the Member State in which the user resides.

Distributors: www.3pp-eifu.com/#distributors

Label Symbols: All of the symbols used on the product label or product literature are defined in the 3-Point Products Symbols Legend here: www.3pp-eifu.com/#symbols



3-Point Products, Inc.
118 Log Canoe Circle
Stevensville, MD 21666 USA
www.3pointproducts.com
+001.410.604.6393 



Sallmann Medical GmbH
Alois-Stockinger-Str.16
5020 Salzburg Austria
service@sallmann-medical.eu



Swiss AR Services GmbH
Industriestrasse 47
CH-6300 Zug