



SCAN TO  
LEARN MORE

3pointproducts.com

**3point**  
products®

## GEL MATE® Silicone Gel Sheets



**Gel Mate** medical grade silicone gel sheets are a first choice for treatment of post-op or post-injury scars. Gel Mate is comfortable over scars or skin abrasions and can be removed without disturbing delicate, healing skin.

**Smooth BEIGE** sheets features an anti-bacterial nylon fabric backing to prevent the sheet from adhering to clothing and helps keep the gel clean. **CLEAR** sheets make it easy to view changes in the scar tissue and is less apparent on facial and neck scars.


### Features and Benefits



Only 2mm thick, Gel Mate provides excellent adherence and stretches for compression and occlusion to help flatten and heal new or old scars.




Translucent gel sheets contour over small finger joints and make it easy to view changes in the scar tissue. Trim to size larger than the scar for optimal coverage and best results.

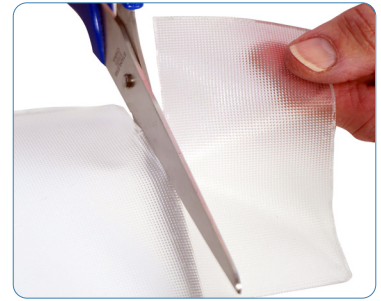

4" x 6" (10 x 15 cm) sheets can be trimmed to size and shape. Packages of 1 or 5 sheets.

## Silicone Gel Tubes

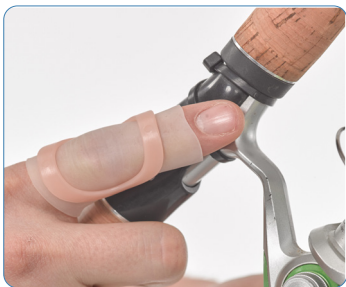

Medical grade mineral oil enriched Gel Tubes ease pressure and friction on fingers and toes. They are ideal under finger orthoses for improved comfort.

Protect and cushion finger lacerations, ulcerations and enlarged joints. Ideal on toes to cushion corns, hammertoes and more.



One size fits most. Flexible tubes are 1/2" (1.5cm) wide by 1 5/8" (4cm) long. Washable and reusable. 5 per package.



SCAN TO  
LEARN MORE

3pointproducts.com



# GEL MATE® Silicone Gel Sheets

## Directions for Use

Use scissors to trim the Gel Mate sheet to size at least one half to one inch larger than the scar. Wash and dry skin over and around the scar. Peel the backing from the gel sheet and apply the adhesive side over the scar. Stretch lightly as needed to achieve good contact over the scar.

## Care

Suggested wear time is 10 to 23 hours per day. Remove once a day to wash with water and mild soap. Place on the plastic cover and allow to air dry. Replace when gel loses its adherence. Daily washing may extend use to a month or more.

## Caution

**Do not use on open wounds.** Use after stitches or closures are removed. Not a sterile product. If rash or discomfort occurs, discontinue use & contact your health care provider.

**Keep out of reach of children & pets.**

## FAQs

**What's the difference between the BEIGE and CLEAR sheets?** The medical grade silicone gel is the same in both sheets. The BEIGE is covered on one side with a soft nylon and spandex fabric. CLEAR sheets have a translucent film cover.

**Can the sheets be cut to make different sizes?** Yes, the sheets can be cut to size or shape. Cut the sheets with the protective backing in place and then peel the backing to apply the gel.

**What are Gel Mate and the Gel Tubes made from?** Gel Mate Smooth BEIGE and CLEAR are made with 100% medical grade silicone gel. Gel Tubes are made from a medical grade mineral oil gel. No natural latex rubber was used in the manufacturing of these medical products.

**How long will Gel Mate and Gel Tubes last?** With proper care and frequent washing, Gel Mate sheets and Gel Tubes can be expected to last up to a month or more. Any debris in the gel will affect the adhesive property.

## To Order from 3-Point Products

Health Care Providers

888-378-7763

Mon-Fri 8:30am - 4:30pm EST

service@3pointproducts.com

## Online Ordering

Patients & Consumers

www.shopoma.com



Scan Here