

# 3-Point Products,<sup>®</sup> Inc.



## Instructions For Use - English (EN)



[www.3pp-eifu.com](http://www.3pp-eifu.com)

**Description:** Oval-8 Finger Splints are a molded plastic device designed to fit on a digit for support, protection and alignment. They are available in 14 sizes to accommodate a range of finger sizes. Oval-8 Finger Splints are waterproof and can be worn in wet conditions.

**Intended Use:** Oval-8 Finger Splints are designed to apply three points of control around a digit to hold a knuckle straight, prevent or limit movement for protection and to allow healing. The device can be used to manage multiple finger conditions depending on where and how it is worn on the digit.

**Precautions:** Not for use on open wounds. Not for use if swelling is present. Not for use on toes. The device may cause irritation or pressure on the digit. Discontinue use if irritation or pressure occurs. Contact your health care provider if irritation persists. Follow cleaning guidelines on a regular schedule. **Caution:** Not to be worn by children under 5 years old. May cause choking if swallowed.




**Direction:** How to wear Oval-8 Finger Splints to manage different finger conditions.


### 1. Boutonniere Deformity




### 4. Mallet Finger

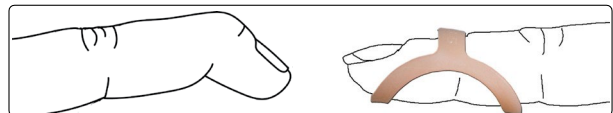


### 2. Crooked Finger

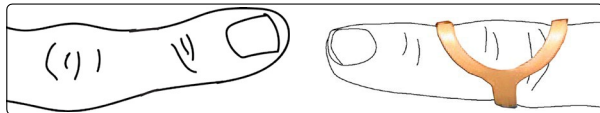


### 5. Swan Neck Deformity

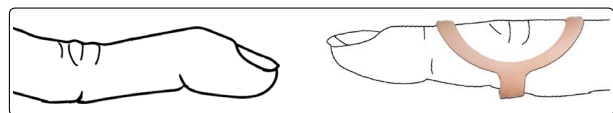


### 3. Trigger Thumb

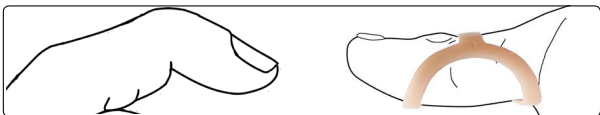


### 6. Trigger Finger

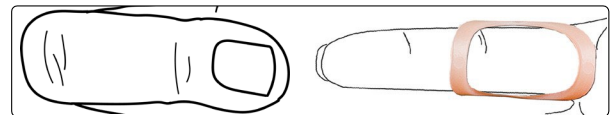

**Note:** Oval-8 splints fit tighter or looser depending on which end is put on the finger first. The plus + sign indicates the looser end.

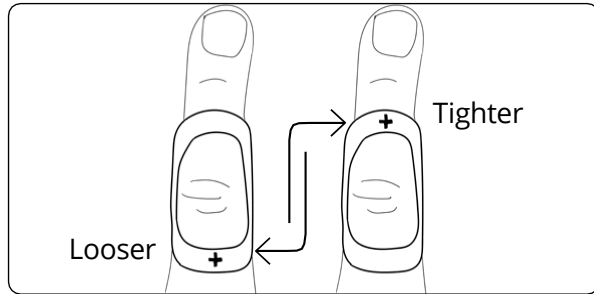

Oval-8 Beige and Transparent

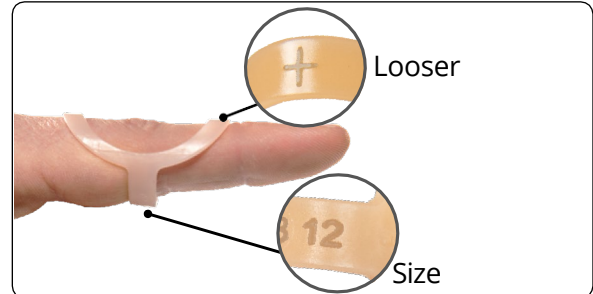

Features appear in both colors

**Measuring for an Oval-8 Finger Splint:** Instructions for measuring digits for the correct size of product can be found on pages 3 & 4.

**How Supplied:** Oval-8 Beige is manufactured with pure polypropylene. Oval-8 Clear are made with copolyester. The product is latex-free. Do not use if the device is damaged. Store in a cool, dry place. It is available in 14 sizes, packaged in various quantities. Packaging options are on page 4.

**Cleaning:** Hand wash with soap and water, and air dry the product regularly. Disinfect the product with 70% isopropyl alcohol or a 10% bleach solution.

**Disposal:** The product is intended to be reusable on the same person multiple times. Discard the device if it shows signs of excessive wear and replace with a new product as necessary.


**When to Consult a Health care Professional:** Consult a health care professional if experiencing any adverse reaction to the device including skin reaction or changes in condition.

**Notice to User and/or Patient:** In the event of an adverse incident related to the device, remove the device and report the incident to the distributor and the Authorized Representative in the Member State in which the user resides.

**Distributors:** [www.3pp-eifu.com/#distributors](http://www.3pp-eifu.com/#distributors)

**Label Symbols:** All of the symbols used on the product label or product literature are defined in the 3-Point Products Symbols Legend here: [www.3pp-eifu.com/#symbols](http://www.3pp-eifu.com/#symbols)



3-Point Products, Inc.  
118 Log Canoe Circle  
Stevensville, MD 21666 USA  
[www.3pointproducts.com](http://www.3pointproducts.com)  
+001.410.604.6393 



Sallmann Medical GmbH  
Alois-Stockinger-Str.16  
5020 Salzburg Austria  
[service@sallmann-medical.eu](mailto:service@sallmann-medical.eu)



Swiss AR Services GmbH  
Industriestrasse 47  
CH-6300 Zug

The Original






Finger Splint

### Measuring your Finger(s) for an Oval-8 Finger Splint

Oval-8 Finger Splints are a medical device and a snug fit generally provides the most effective results. The correct size depends on what the finger problem is and where the band will be worn. Please follow instructions

#### You will need:

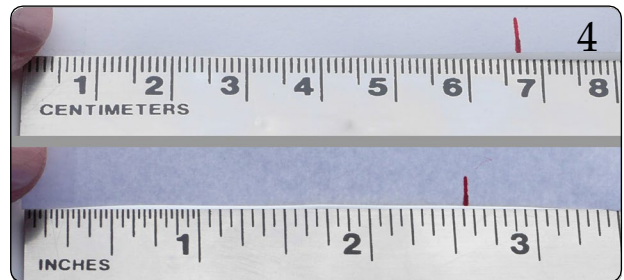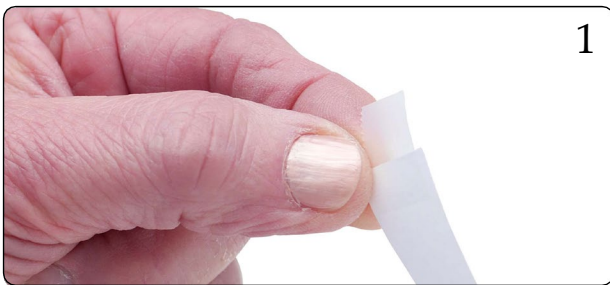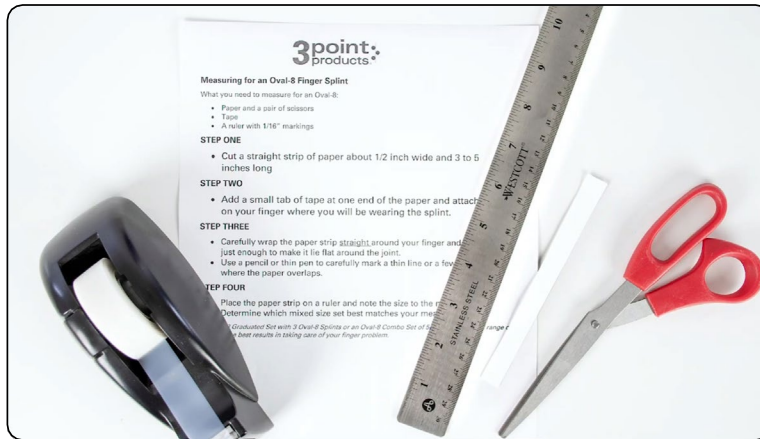

The Original



Finger Splint

Oval-8 Sizing & Packaging Charts

Oval-8 Size Chart		
Oval-8 Size	Measurement Inches	Measurement Centimeters
2	1-3/4"	4.3
3	1-13/16"	4.5
4	1-15/16"	4.95
5	2-1/6"	5.27
6	2-3/16"	5.7
7	2-5/16"	5.86
8	2-7/16"	6.14
9	2-1/2"	6.35
10	2-11/16"	6.82
11	2-13/16"	7.09
12	2-15/16"	7.4
13	3"	7.65
14	3-3/16"	8.09
15	3-1/4"	8.2

Oval-8 Packaging Chart							
Oval-8 Size	One (1) per package	Three (3) per package	Five (5) per package	Multi-Size Packs three (3) sizes	Oval-8 Kit Clinic box with 44 splints Available only in beige	Quantity per size	
2	✓	✓	✓	2-3-4	Size 2 & 15	1	
3	✓	✓	✓	4-5-6		Size 3-4 & 13-14	2
4	✓	✓	✓	6-7-8		Size 11 & 12	3
5	✓	✓	✓	8-9-10		Size 5 & 10	4
6	✓	✓	✓	10-11-12		Size 6-7-8-9	5
7	✓	✓	✓	13-14-15	Sizing Set one (1) each size - 14 total	2-3-4-5-6-7-8-9-10-11-12-13-14-15	
8	✓	✓	✓	Multi-Size Packs five (5) sizes			
9	✓	✓	✓				
10	✓	✓	✓				
11	✓	✓	✓				
12	✓	✓	✓				
13	✓	✓	✓	2-3-4-5-6			
14	✓	✓	✓	5-6-7-8-9			
15	✓	✓	✓	8-9-10-11-12			
				11-12-13-14-15			

All packages may not be available from all sellers